


EARWARE

SRM-2000



WIRELESS BLUETOOTH MICROPHONE

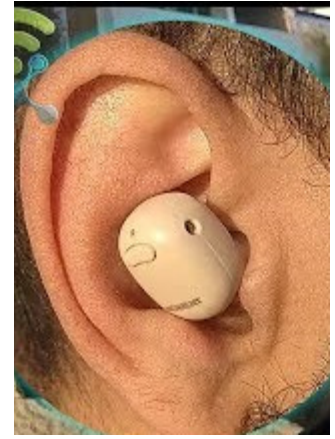

WIRELESS EARLEVEL RECEIVER

- Communicate with Assistants/Parents in the Sound Room for VRA Testing
- Coordinate Testing Procedures for Best Results
- One-Way communication from Tester to Assistant
- Ear Level Hands-Free –NO WIRES!
- Interference Free –Excellent Sound Quality
- Rechargeable Internal Batteries

Miniature wireless communication system consists of a lapel transmitter and ear level ear-pierce that is compact and easy to use offering hands-free communication to an assistant or caregiver in the sound room during VRA testing. Miniature wireless transmitter can be worn or placed on the desk for clear communication. Comes complete with AC charger for up to 4 hours of continuous use before units needs recharging. Wireless Bluetooth transmission offers excellent sound quality over a 30 foot range

PEHRATEK PRODUCTS

9285 Pierson Lake Drive
Chaska, MN 55318
866-470-3532

FIRST TIME PAIRNG EARWARE 2000

1. Charge both microphone and ear piece for 6 hours
2. Set mode switch on microphone to "A"
3. Hold down microphone power button until blue light blinks fast
4. Remove earpiece from charging case and hold down button until blue light blinks fast
5. Keep both units within one foot of each other for pairing
6. When booth blue lights go steady they are paired
7. These devices do not need to be repaired for continued use
8. Follow voice prompts for pairing confirmation

RECHARGE JACK



TO RECHARGE MICROPHONE-PLUG IN CHARGER TO TOP JACK

TO RECHARGE EARPIECE –PLACE INTO BASE

VOLUME CONTROL

To increase volume levels –press ON/OFF button once to increase the volume one step to reach the desired listening level. The RED led will flash for each volume step change

PEHRATEK PRODUCTS

9285 Pierson Lake Drive

Chaska, MN 55318

866-470-3532